
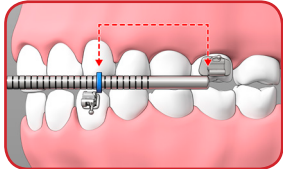
System Installation Instructions

1 Choose Appliance Size

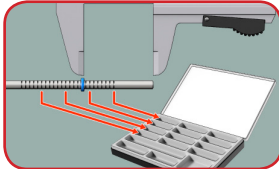
Use a gauge to measure the length between the mesial side of the maxillary tube. Move ring until the distal side of the bracket on the mandibular canine.


Use a calliper or a ruler to measure the distance and then choose the correct pre-adjusted system. Four pre-adjusted sizes are available.

2 Adjustment of Mandibular Section

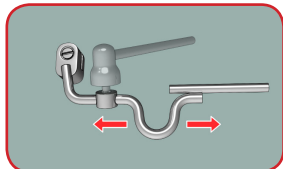A laser marking indicates right or left of the appliance's mandibular section.

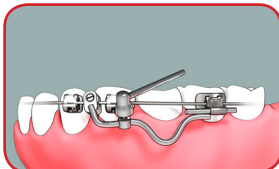
With the help of an Omega loop, adjust the mandibular side to ensure patient comfort and the correct position of the lock.

Verify your measurements by inserting metal rod into the .045" EOT tube on mandibular band. Lock should be positioned between the canine and premolar.

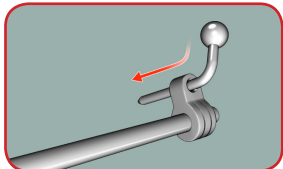3 Preparation of Maxillary Section


The ball and hook hinge is bent at a 90° angle and inserted into the main appliance. The ball should be located on the buccal side.

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
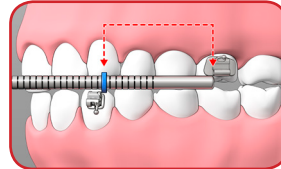
EN

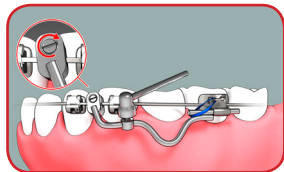
4 Final Placement

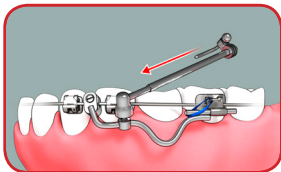Insert mandibular section into the .045 tube until it stops. Position the lock between canine and premolar, block it and screw. The section is ligated to the molar band with a metal 0.12 ligature in addition to an elastomeric ligature.

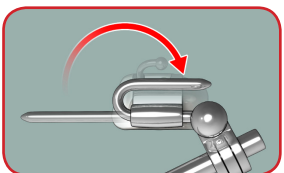The maxillary tube is insert into the mandibular rod. With the patient's mouth wide open, rotate completely the entire appliance.

Insert the maxillary ball and hook hinge from the distal side of the upper 6's EOT tube. The mesially protruding part of the ball and hook hinge is then folded back from mesial to distal above the tube. The system is thus fixed in place and operational.

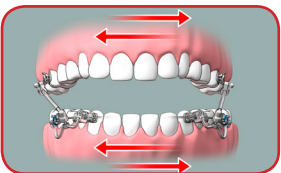
5 Repeat Instructions for Opposite Side

Repeat steps 1 through 4 for the opposite side. The system is in place.

6 Verify Functionality

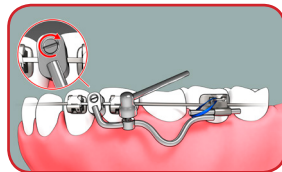
Verify that the mouth can open completely.

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